


FetaLink is an application used by clinicians to monitor maternal and fetal data during the different phases of labor. FetaLink allows for data to be displayed in a meaningful and useful format to clinicians in the ambulatory and inpatient settings.

Documentation Integration

Ambulatory practices will document the Fetal Monitoring episode after the Non-Stress Test order has been placed from the WE Quick Order Mpage by entering a Reason for Monitoring and any necessary nursing annotations in FetaLink.


➤ Documentation in FetaLink

- Documenting Reason for Monitoring:
 - Associate the patient to the monitor.
 - From the Census view, select the column **Reason for Monitoring**.
 - Associate the patient to the correct encounter.
 - Select **Reason for Monitoring**.
 - Select the reason for monitoring from the provided list.
 - Select OK.

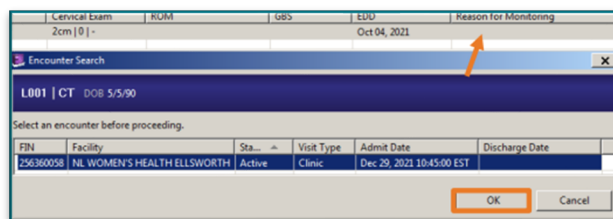

NOTE: During the monitoring episode, address any alerts and create an annotation.

- Creating an Annotation for an alert:
 - Select the **paper clip** icon.
 - Select options from the quick picks.
 - If the options are not appropriate, free text in the annotation box.

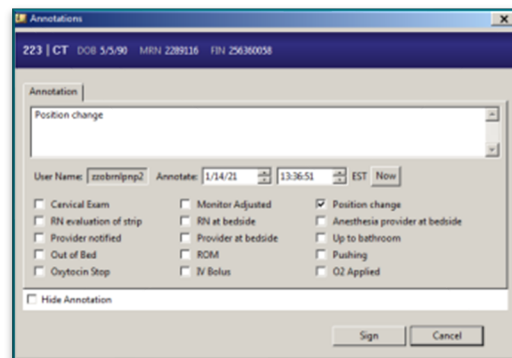

➤ Documentation in PowerChart from FetaLink

- Annotations flow automatically to the FHR Annotations Field found in the Antenatal Testing Band of Interactive View and I & O.

		01/14/2021	
		13:44 EST	13:36 EST
Fetal Monitoring Annotations			
Fetal Monitoring Annotations		Position change [2]	

➤ Documentation from PowerChart to FetaLink

- The provider will document their exam/interpretation in the Antenatal Testing Band in Interactive View and I & O.
- This information will display in FetaLink as an Annotation.

➤ Finalizing the Fetal Monitoring Episode




- When the fetal monitoring episode is complete, the patient will be taken off the monitor and the episode will be finalized.
- Select **Finalize**.
- Review/update the **Reason for Monitoring** and select **Yes**.

➤ Single Patient Task List

- After the fetal monitoring episode has been finalized, review the patient's Single Patient Task List.
- From the drop down, select **Done**.
- Complete the form, as appropriate, and select the **green checkmark** to sign the form.

➤ **Antenatal Testing Note**

- From the Prenatal MPage, select **OB Antenatal Testing**.
- If being initiated by nursing, nursing will review the information and save the note.
- Navigate to Notes from the table of conecnts, and locate the note initiated by nursing and select the person icon.
- Providers are able to review the note in their Message Center prior to signing.

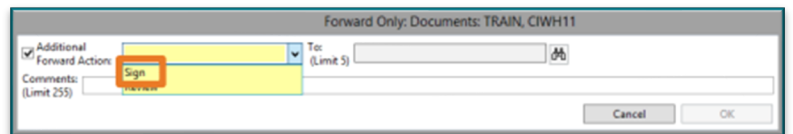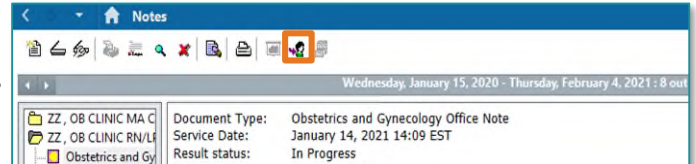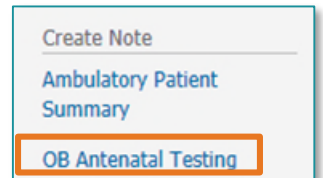

➤ **Viewing the Fetal Monitoring Episode**

- From the Prenatal Mpage, select **Fetal Monitoring**.
- Select the episode to view and select **Launch**.
- To view a PDF form of the fetal monitoring episode, select **Media Gallery**.
 - Locate the fetaLink PDF and click the episode to view.

