

# ECG Management - Phillips ECG Cart Workflow

## User Training


**\*\*\*\*\*Order driven workflow\*\*\*\*\* CV ECG or CV ECG Pediatric MUST be ordered first. Exception - If patient is emergent and not registered follow current workflow.**

**STEP 1: Go to the ECG cart. Press the ID button.**




**STEP 2: Select the Worklist tab.**


- Refresh the worklist on the cart by clicking the drop down, select **POWERCHART INBOX**.
- Press **Get Worklist**.

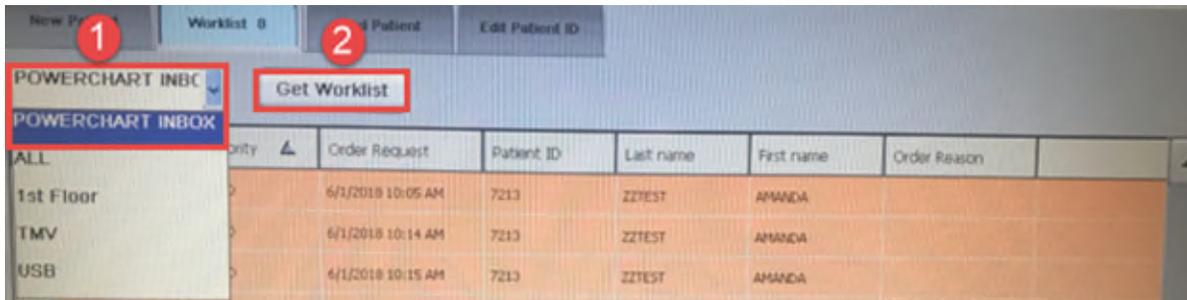


**STEP 3: Select the patient from the list and acquire the ECG.**

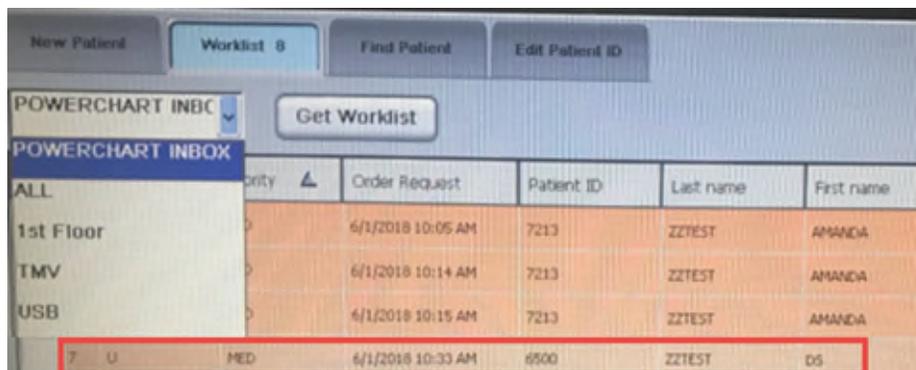

**STEP 4:** Connect the leads to the patient and press the “ECG” button on the cart to acquire the ECG.

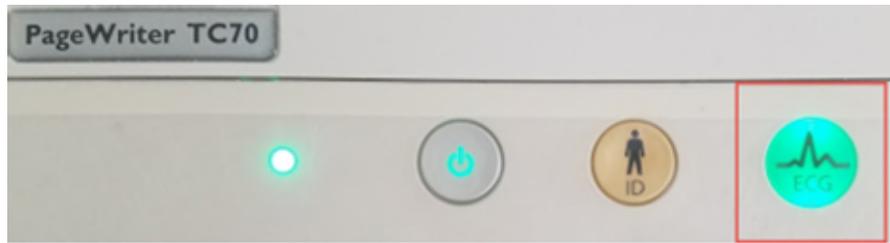

**STEP 5:** Once the ECG has been acquired push the Transfer or Save button to transmit the ECG to Cerner.

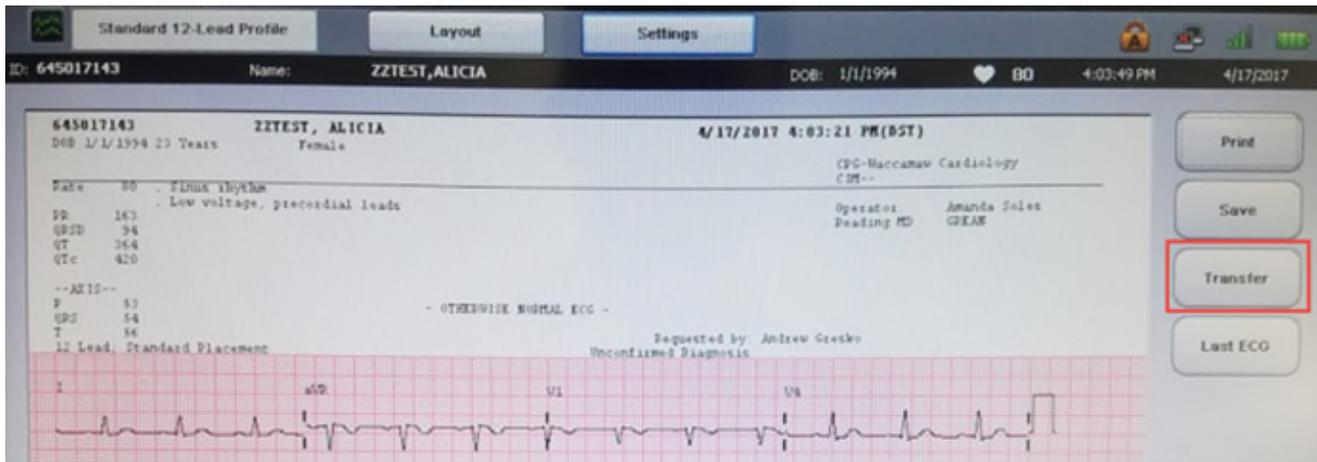

**\*\*\*Please ensure the quality of the ECG Tracing is free of artifact and wander prior to transferring the study to Cerner.\*\*\***