


# ECG Management™ - Welch Allyn ECG Cart Workflow


## User Training

**\*\*\*\*\*Order driven workflow\*\*\*\*\* CV ECG or CV ECG Pediatric MUST be ordered first. Exception - if patient is emergent and not registered, follow current workflow.**

**STEP 1: Go to the ECG Laptop. Start a Resting ECG.**

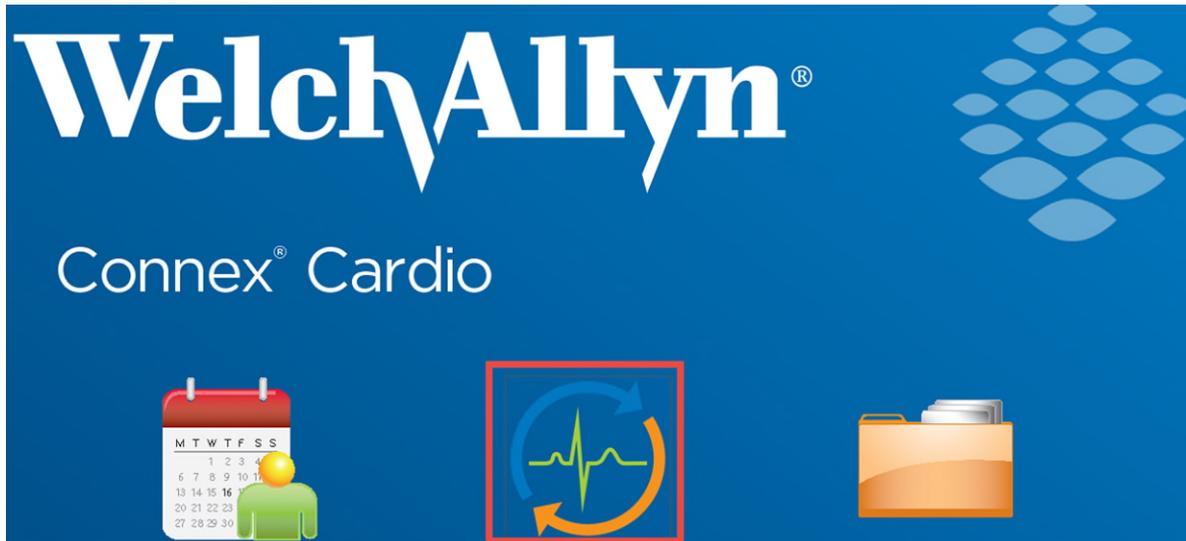

**STEP 2: Select the MWL then select the Search button.**

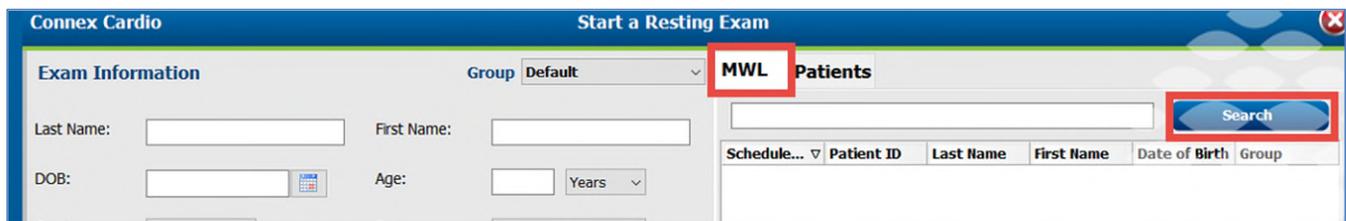

Orders will be returned for 5 days prior and 1 day forward, just like they appear on the ECG Management worklist. Completed exams will not be returned on the worklist. **Be sure to validate the date/time** of the order you select is correct.

Validate patient information. It should be the same as what appeared on the ECG Management worklist.

**STEP 3:** Select the patient from the list and press Start Exam.

<input type="text"/>			Search
Patient ID	Last Name	First Name	Date of Birth
2289290	TESTING	ABIGAIL	
0624544	Testing	Alex	

Start Exam	Clear	Exit
------------	-------	------

**STEP 4:** Connect the leads to the patient and press the ECG button on the cart to acquire the ECG.



**STEP 5:** Once the ECG has been acquired push the Save button to transmit the ECG to Cerner.



**\*\*\*\*Please ensure the quality of the ECG Tracing is free of artifact and wander prior to transferring the study to Cerner. \*\*\*\***