



The QTcB field will display the Bazett formula result within the ECG Viewer. Providers and staff can display 2 additional QTc formulas in the ECG viewer if needed. Follow the steps below to add QTc formulas to view.

Setting the QTc Preferences

STEP 1: From PowerChart/FirstNet, click the **Cardiovascular** button in the toolbar.

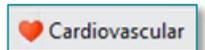

STEP 2: Click the **Cardiovascular** tab in the top toolbar.

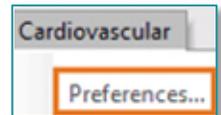


STEP 3: Select **Preferences**.

STEP 4: Click **PowerChart ECG**.

STEP 5: Click the checkbox to the left of **Show** to select desired **QTc formula**.

- Click **Apply**.
- Click **OK**.

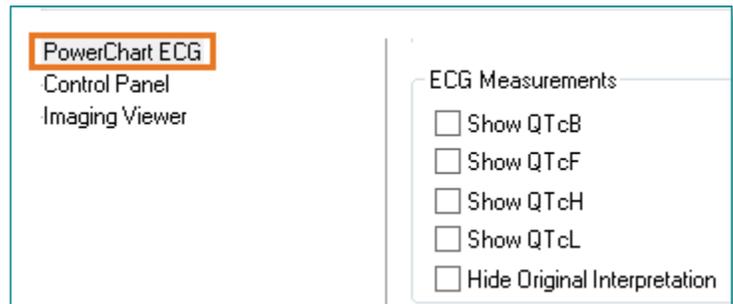

NOTE: Two additional QTc formulas can be displayed in the ECG Viewer additional to the QTcB.

Changing Measurement Values Within the ECG Viewer.

STEP 1: Use the calipers within the ECG Viewer to set the Ventricular Rate, PR Interval, QT Interval, and QRS Interval.

STEP 2: Measurement changes within the discrete data boxes will show in bold font.

NOTE: The QTcB is NHL's source of truth for the QTc formula.

Acquired: 11/29/2016 16:07:27

Vent rate	66	BPM
PR int	173	ms
QRS dur	122	ms
QT/QTcB	561	590 ms
P-R-T axes	68	46 88
Avg RR	903	ms
QTcF	580	ms
QTcH	572	ms

