



The QTcB field will display the Bazett formula result within the ECG Viewer. Providers and staff can display 2 additional QTc formulas in the ECG viewer if needed. Follow the steps below to add QTc formulas to view.

Setting the QTc Preferences

STEP 1: From PowerChart/FirstNet, click the **Cardiovascular** button in the toolbar.

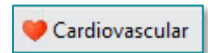

STEP 2: Click the **Cardiovascular** tab in the top toolbar.

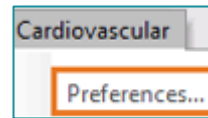


STEP 3: Select Preferences.

STEP 4: Click PowerChart ECG.

STEP 5: Click the checkbox to the left of **Show** to select desired QTc formula.

- Click Apply.
- Click OK.

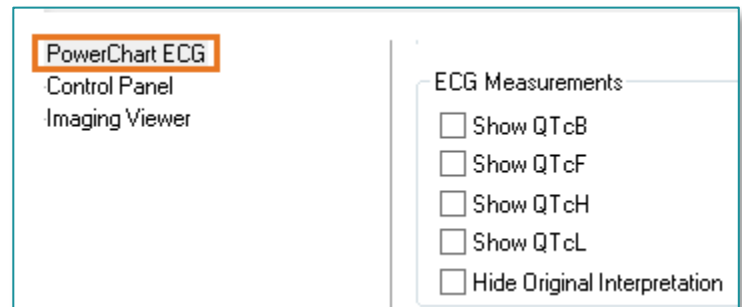

NOTE: Two additional QTc formulas can be displayed in the ECG Viewer additional to the QTcB.

Changing measurement values within the ECG Viewer.

STEP 1: Use the calipers within the ECG Viewer to Set the Ventricular Rate, PR Interval, QT Interval, and QRS Interval.

STEP 2: Measurement changes within the discrete data boxes will show in bold font.

NOTE: The QTcB is NHL's source of truth for the QTc formula.

Acquired: 11/29/2016 16:07:27			
Vent rate	66	BPM	
PR int	173	ms	
QRS dur	122	ms	
QT/QTcB	561	590	ms
P-R-T axes	68	46	88
Avg RR	903	ms	
QTcF	580	ms	
QTcH	572	ms	

