

FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in the acute and ambulatory settings. There may be times that the association time of the monitor may need to be modified.

Modifying the Association Time

STEP 1: From the toolbar in the patient chart, select the **Monitor Association** button.

- The **Monitor Association** window will appear.
- The association start date and time that was previously established displays.

STEP 2: Within the Monitor Association window, select **Modify**.


STEP 3: Update the time field with the correct association time, changing the date as necessary.

STEP 4: Select **Apply** to commit the changes.

- The updated association time displays.

NOTE: The association date/time can be back dated up to one week.

NOTE: If the user doesn't remember what time the strip started, review the Annotation Summary.

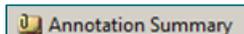

Annotation	Date/Time
 CMON-TEST-01 has been connected.	Aug 25, 2021 16:19:45 EDT
 CMON-TEST-01 has been disconnected.	Aug 25, 2021 16:19:20 EDT
 CMON-TEST-01 has been associated.	Jun 24, 2021 15:56:09 EDT

NOTE: Selecting **Cancel** will close the window and keep the current association start date/time.

