


FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in the acute and ambulatory settings. There may be times when you want to view multiple tracings.

View Multiple Tracings


STEP 1: To view more than one strip at a time, click the box next to the desired monitor location, from the Census View.

Obstetrics NL Women's Health Blue Hill NL Women's Health Ellsworth All Locations				
	Monitored Location	Fetal Monitor	Patient	FIN
<input type="checkbox"/>	MCH OB1			
<input type="checkbox"/>	MCH OB2			
<input type="checkbox"/>	MCH OB3			
<input checked="" type="checkbox"/>	MCH OB4	CMON-TEST-04	CT	256328071
<input checked="" type="checkbox"/>	MCH OB5	CMON-TEST-01	FT	256418070
<input type="checkbox"/>	MCH OB6			

STEP 2: Select View.

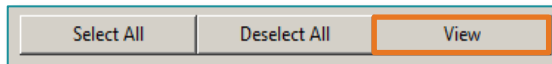

STEP 3: Multiple patient view displays as **Monitored Locations**.

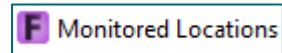

- Multiple views are different than the Central Monitoring and Alert views.

STEP 4: To view a single monitor when in the Multiple Views, click the **Expand** icon of the desired monitoring location to view a single location.



- Clicking the icon in the corner again will return you to the Multiple View.

STEP 5: To exit Multiple Views, unselect the rooms in the **Census View**.