


FetaLink is an application used by clinicians to monitor maternal and fetal data during the different phases of labor. FetaLink allows for data from medical devices to be displayed in a meaningful and useful format to clinicians in both ambulatory and acute settings.

Viewing Fetal Monitoring Episodes

➤ Fetal Monitoring

- From the Fetal Monitoring component, select the episode desired to view.
- Select **Launch**.
- FetaLink will launch and the fetal monitoring strip can be viewed within FetaLink.

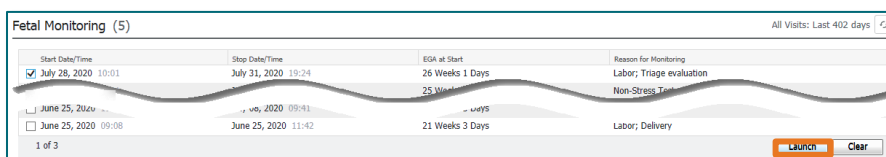

➤ Media Gallery

- To view a PDF of the fetal monitoring episode, select the **Media Gallery** component.
- Select the desired episode to view labeled FetaLink PDF.
- Select **View**.
- A pop-up for Media Viewer will display. Highlight the PDF image in the view panel.
- The PDF of the fetal monitoring strip will be displayed in the lower pane of the Media Viewer.

