

This flyer outlines how to assign exams from Enterprise Imaging (EI) Desktops to No Read-Cancel.

Assign study to No Read-Cancel



STEP 1: From the Task list, right-click the patient.

STEP 2: Select Assign.

STEP 3: Select No Read-Cancel.

STEP 4: Click the Arrow move to Assign to.

STEP 5: Select Assign.

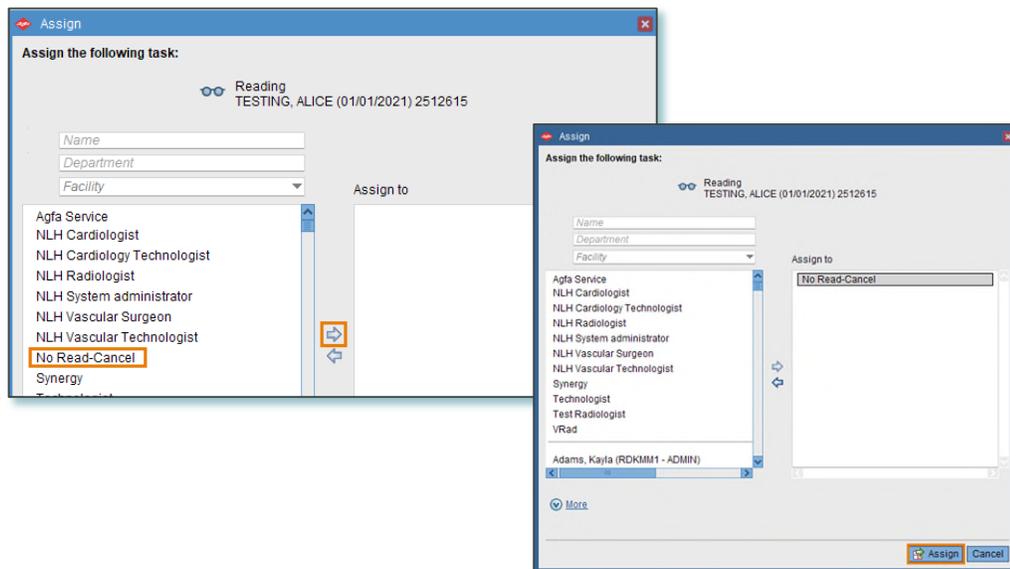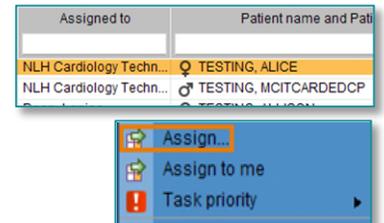

NOTE: When a Radiologist is not required to read a study, the technologist performing the acquisition step will assign the No Read-Cancel to remove from the Radiologist's To Do list