

## Activities Overviews allows staff to add exam lists in Enterprise Imaging (EI).

### Activities Overviews

Enterprise Imaging (EI) enables staff to view ordered exams specific to location, modality, and status. Follow the steps below to add a location or modality to your **Activities Overviews** list.


**STEP 1:** Login to the appropriate EI Desktop for your role.

**STEP 2:** Select **List area**.

**STEP 3:** Select **Activities overviews**.

**STEP 4:** Select the radio button in **Default** column for the Exam Tasklist used most.

**NOTE:** To add additional Exam Tasklists, check the box in the Show in dropdown list.

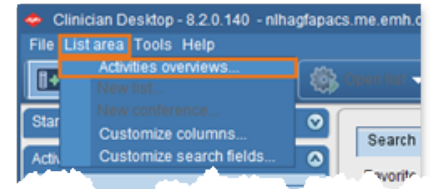

Manage Activities Overviews

Overview

75 results

Show in drop-down list	Default	Personal	Code	Name	Available for	Active from	Active to
<input type="checkbox"/>	<input type="radio"/>		ACTOV4819	Spectrum Remote Dx Weeke...	System, NLH Radiologist, NL...	09/29/2022	
<input type="checkbox"/>	<input type="radio"/>		ACTOV14002	Spectrum FPPE	NLH Radiologist, System ad...	02/16/2024	
<input type="checkbox"/>	<input type="radio"/>		ACTOV15	BHMH Cardiologist Reading ...	System, NLH Cardiologist, N...	09/29/2022	
<input type="checkbox"/>	<input type="radio"/>		ACTOV205	ARG Technologist Modality ov...	NLH Imaging Technologist, N...	10/01/2022	
<input checked="" type="checkbox"/>	<input checked="" type="radio"/>		ACTOV4826	Clinician Workflow	System, NLH Clinicians, NLH...	07/06/2023	
<input type="checkbox"/>	<input type="radio"/>		ACTOV4814	Spectrum SJH MG Rad Task	System, NLH Radiologist, NL...	09/29/2022	
<input type="checkbox"/>	<input type="radio"/>		ACTOV5801	EMMC Sonographer Overview	NLH Imaging Technologist, N...	10/01/2022	
<input type="checkbox"/>	<input type="radio"/>		ACTOV13	Mercury Cardiovascular Reading T...	System, NLH Cardiologist, N...	09/29/2022	
<input type="checkbox"/>	<input type="radio"/>		ACTOV17	SVH Radiologist Reading Task	NLH Radiologist, NLH Syste...	09/29/2022	
<input type="checkbox"/>	<input type="radio"/>		ACTOV204	Mercury Technologist Modality o...	NLH Imaging Technologist, N...	10/01/2022	

New Duplicate Edit Delete

Close

**STEP 5:** Click **Close** in the bottom right corner of the screen to return to the **Task List** – “To do.”

**STEP 6:** Click the checkbox(es) to the left of the modality in the **Details** dropdown to open the **Task List** for the selected modalities.

