

Clinician Workflow and adding imaging studies to Ortho On Call conference in Enterprise Imaging (EI).

Clinician Workflow under Activities Overviews for Adding Studies to Conferences Folder

Enterprise Imaging (EI) enables clinicians to add studies to conferences named **Ortho On Call**. Follow the steps below to add studies and **Clinician Workflow** to the **Activities Overviews** list.

STEP 1: Login to the **Clinician EI Desktop**.

STEP 2: Click **List** area.


STEP 3: Click **Activities overviews** in the dropdown box.

STEP 4: Select **Clinician Workflow** by checking the box in the **Show in dropdown list** column.

STEP 5: Click **Close** in the bottom right corner of the screen to return to the **Collection list** – “My list”.

STEP 6: Click **dropdown arrow** in Activities overviews.

STEP 7: Select **Clinician Workflow**.

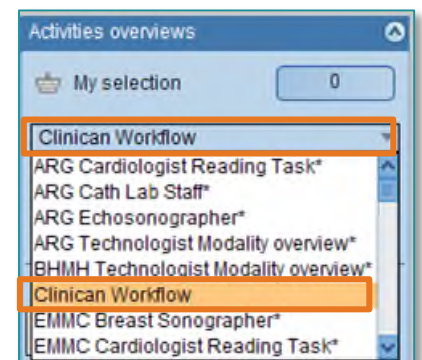

Follow the steps below to add studies to **Ortho On Call** conferences.

STEP 1: Search study of interest.


STEP 2: Right-click the study.

STEP 3: Click **Add to conference**.

STEP 4: Select **Ortho On Call**.

STEP 5: Click **At the next session**.

STEP 6: Click **OK**.

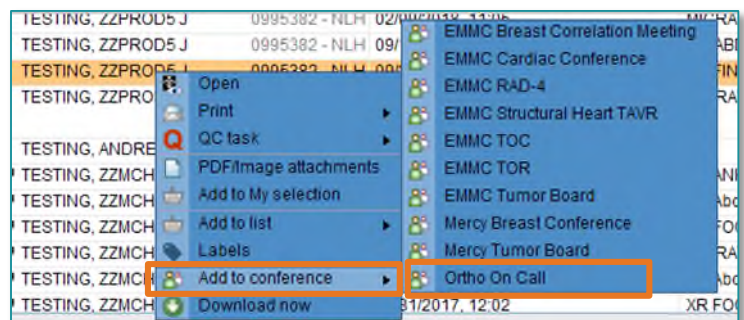

NOTE: Studies will be automatically removed the next day.