

From the Office of Clinical Informatics

Activities overviews

m My selection

Clinican Workflow

ARG Cath Lab Staff*

Clinican Workflow

ARG Echosonographer*

EMMC Breast Sonographer

ARG Cardiologist Reading Task

ARG Technologist Modality overview* BHMH Technologist Modality overview*

EMMC Cardiologist Reading Task*

Enterprise Imaging (EI) Clinician Workflow and Ortho On Call Conference

January 3, 2024

ω

0

Clinician Workflow and adding imaging studies to Ortho On Call conference in Enterprise Imaging (EI).

Clinician Workflow under Activities Overviews for adding studies to conferences folder

Enterprise Imaging (EI) enables clinicians to add studies to conferences named **Ortho On Call**. Follow the steps below to add studies and **Clinician Workflow** to your **Activities Overviews** list .

STEP 1: Login to the Clinician EI Desktop.

STEP 2: Click List area.

STEP 3: Click **Activities overviews** in the drop-down box.

STEP 4: Select Clinician Workflow by checking the box in the Show in

dropdown list column.

<u>STEP 5</u>: Click Close in the bottom right corner of the screen to return to

the Collection list - "My list".

STEP 6: Click **drop down arrow** in Activities overviews.

STEP 7: Select Clinician Workflow.

Follow the steps below to add studies to Ortho On Call conferences.

STEP 1: Search study of interest.

STEP 2: Right Click on the study.

STEP 3: Click Add to conference.

STEP 4: Select **Ortho On Call**.

STEP 5: Click At the next session.

STEP 6: Click OK.

0995382 - NLH 09/ B EMMC Breast Correlation Meeting TESTING, ZZPROD5 J NILL OOF 8 EMMC Cardiac Conference TESTING, ZZPRODE Open 8 EMMC RAD-4 TESTING, ZZPRO 8 EMMC Structural Heart TAVR TESTING, ANDRE Q QC task 8 EMMC TOC 8 EMMC TOR PDF/Image attachments TESTING, ZZMCH TESTING, ZZMCH 🖮 Add to My selection 8 EMMC Tumor Board TESTING, ZZMCH m Add to list 8 Mercy Breast Conference TESTING, ZZMCH Labels 8 Mercy Tumor Board TESTING, ZZMCH 🐣 Add to conference Ortho On Call TESTING, ZZMCH (Download now 1/2017, 12:02

NOTE: Studies will be automatically removed the next day.