

This flyer outlines how to assign exams from Enterprise Imaging (EI) Desktops to VRad/Synergy.

Assign study to VRad/Synergy



STEP 1: From the Task list right-click the patient.

STEP 2: Select Assign.


STEP 3: Select either VRad or Synergy.

STEP 4: Click the Arrow move to Assign to.

Assigned to	Patient name and Or
	TESTING, ZZPROD23
NLH Vascular Technolog...	TESTING, Bruce
NLH Cardiology Technol...	TESTING, ZZPROD1

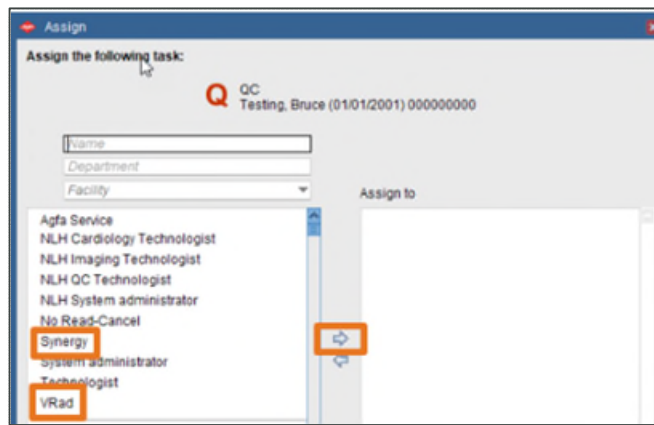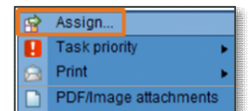

STEP 5: Select Assign.

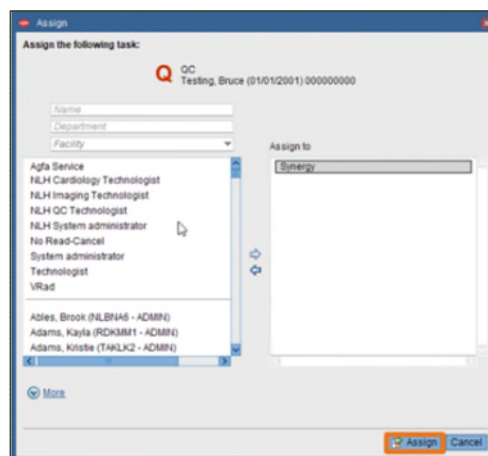

NOTE: All VRad studies need a Radiologist final reading.

For questions regarding process and/or policies, please contact your unit's Clinical Educator or Clinical Informaticist. For any other questions please contact the Customer Support Center at:
207-973-7728 or 1-888-827-7728.