

From the Office of Health Informatics Enterprise Imaging (EI) XERO Workflow – EP Lab at EMMC

July 3, 2025

Enterprise Imaging (EI) XERO Workflow for EP Lab at EMMC. This workflow is currently only used at EMMC.

<u>STEP 1</u> :	 Select XERO Workflow EMMC. Link located on EP desktops. 	Northern Light Health. 🏫 🗮 Menu			vanch VM VM Clas van Andreilang Sarvices W Grine Wagstar - Home Care & Hespite Washer Waher Building Sign Update Washer	م		
<u>STEP 2</u> :	Type User name, currently used in IMPAX.	Enterprise Imaging XERO Viewer 8.2.0.140	Enterprise Imaging XERO Viewer 8.2.0.140			Won's On Call Won's White is Landarship WHS WHS Women's Hauth Protocols and Algorithms Women's Hauth Protocols and Algorithms WORKS Lagin MOD without Call Model of Mice State Nanodical FAX What Your Care Opposison Videos		
<u>STEP 3</u> :	Type Password , currently used in IMPAX.	DUser.						
<u>STEP 4</u> :	Select Login.							
<u>STEP 5</u> :	Select the patient needed.	E Search patient	EMMC Reading Cath Lab Staff		⊘ Helo =	Nicole Gray 👻		
<u>STEP 6</u> :	Select desired Test.	EMMC Cam Lab Statt" -	22 tasks, 1 STATI Q Recently construct Pacient/procedure 4 G-3 TESTING PROD1	Patient 10 Date of birth Geno 05/01/2011 Q	Quick search	0, 0 Task creation da		
<u>STEP 7</u> :	Ascend opens.	EVMC Acquisition IR EVMC Cash OC tasks	CLICD Implant	05/01/2011 Q		07/07/2023		
<u>STEP 8</u> :	Enter Study details.	Completed today EMMAC BMMAC B	CLICD Bivent Implant	Ŷ	MARTIN, BRIAN C	07/07/2023,		
<u>STEP 9</u> :	Select To be Read.	My recently viewed studies My recently viewed studies Unread Studies EMMC Cath	6ð	Ŷ	TSOMDES, ALLISON M POLSOM, SARAH E	07/07/2023, 07/07/2023,		
		EMMC Cardiac Conference EMMC Structural Heart TAVR	 ы 	ď	CAWLEY, ELISABETH G	07/07/2023,		
	Study details Patient: BROWN, CYNTHAA M T Study Patient Account number 3919/4734 Account number 3919/4734 Account number 07/07/2023 1104 Avid Account number 07/07/2023 1104 Avid Account number 02/07/2023 1104		w Pur Relean	enauge Root Parlance (top v Ko	en Curringhum *		
	Cele dateIntre O/0702231101 AM Universition Versitive ID C. Contramy Analogophy wir Interversion;200000156; Order description Ulgency Case Lip discharge dateTime Location; performet Procedure (com Bhufty	 Construction of the problem Construction of the problem		NUM: Ordening physician OURAT-UL-AN LELAN M Parpose of study: STEM. CHEST PAN. Semeny: New annay: Mo Recommendations: [Services nuclearity] Seatly data: Parlies and transmit; Seatly data: Parlies and transmit; Report has not been signed	,			

Next

Ok Cancel

Namo	0	Departm	ent	*	Facility	
Norric	4	Departing	ent		raciity	
Users and ass	ignment group	s	1	ssign to		
NLH Cardiol	ogist					
NLH Cardiol	ogy Technologis	it				
NLH Radiolo	ogist _					
NLH System	administrator		\rightarrow			
NLH Vascula	r Surgeon		<			
NLH Vascula	r Technologist					
No Read-Ca	ncel					
Synergy						
			l			
Task priority	Calculated by	system •	•			
Task comment	(optional)					

<u>STEP 10</u>: To search for a particular person, type name in **Assign the following task: Name**.

 NOTE:
 XERO Workflow is unable to do measurements, adjustments, TOMTEC, and Invia are not available.

 Once "To be read" is selected and changes are needed, user must sign into Diagnostic Desktop and assign the study to themselves to make the changes.

For questions regarding process and/or policies, please contact your unit's Clinical Educator. For questions regarding workflow, please <u>place a ticket</u> to Health Informatics. For any other questions please contact the Customer Support Center at: 207-973-7728 or 1-888-827-7728.