

July 2, 2025

XERO Viewer is a medical image viewer that can be launched by an image link in the medical record or from a Web-Browser

Accessing XERO Viewer

- Open the patient's chart in PowerChart, navigate to the Imaging component on a Workflow MPage, and select the imaging study to be viewed.
 - Click the View Image icon.
- NOTE: Image link 🖃 in the EHR (Electronic Health Record) opens directly to the patient and study that is being opened.

Accessing XERO Viewer using a Browser

- Users not needing to use Ascend reporting.
 - Click to open link <u>https://brnagfaweb.me.emh.org/</u>.
- Cardiology and EMMC Vascular workflow to use Ascend reporting.
 - Click to open link <u>https://brnagfaweb.me.emh.org/?returnTo=/workflow/</u>.
- **<u>NOTE</u>**: These links can be added to My Quick Links on the Intranet Home Page.
- **STEP 1:** Enter **Patient Name**, **Date Of Birth**, **Accession number**, and **Study Date (**if known), and **Modality** to search for patient.
- **STEP 2:** Click **Search**.
- <u>STEP 3</u>: Select method of display, **Report and Images**, **Report Only**, or **Image Only**.



- **<u>STEP 4</u>**: Double-click the study.
- **<u>STEP 5</u>**: Select right arrow to show image area.

File Edit View Favorites Tools Help				-		<u>(21</u>
> 🗈 🤒				On	line 🕜 Help	(Fall Logor
Patient Name mouse Patient ID Date Of Birth YYYYMMDD	Accession Study Date V Modality CR V	Search Clear Search for studies				
Filter Patients	Filter Studies	×		List View	Sort by	~
mouse, mickey (M)		ID: AG00011846	DoB: Feb 17, 1983			~ >
🔲 📄 CR - CHEST 1 VIEW	· <u>(</u>)	Acn: AGFA000000000000	May 18, 2018 11:42 AM			

<u>STEP 6</u>: Select arrow on the bottom middle of the screen to scroll between images.



3 🗊

- **<u>STEP 7</u>: Patient Banner** is used to verify patient.
- **<u>STEP 8</u>**: Select the switch series icon in the upper right to switch between series.
- **<u>STEP 9</u>**: Click one of the **Tools** to assist with reviewing images.
- **<u>STEP 10</u>**: Select the ellipsis to change layout display.
- **<u>STEP 11</u>**: Use the **Help** link to assist with any questions.
- **<u>STEP 12</u>**: Click **Logout** once review of image(s) has been completed.



For questions regarding process and/or policies, please contact your unit's Clinical Educator. For questions regarding workflow, please <u>place a ticket</u> to Health Informatics. For any other questions please contact the Customer Support Center at: 207-973-7728 or 1-888-827-7728.