
FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in acute and ambulatory settings. This flyer will discuss the use of FetaLink +.


FetaLink +

FetaLink+ is a mobile application that will be used by OB providers on their IOS or Android mobile devices (phone or iPad). The app is a view only mode for the patient's fetal monitoring.


- From the App store or Google play, select FetaLink+ and install on the device.




- When the app launches, sign in with Cerner login and password at the prompt.

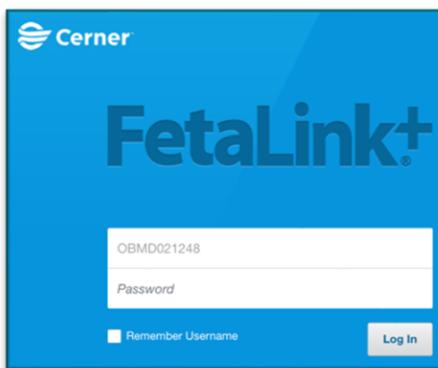


- Filters can be created by location and providers.

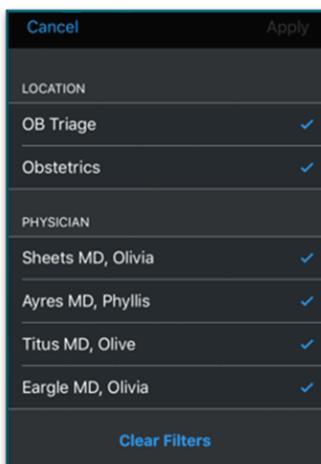

- The patient's names with location. EGA, room number, and provider's name will be displayed based on location. [Note: Content displayed below contains fictitious patient information.]

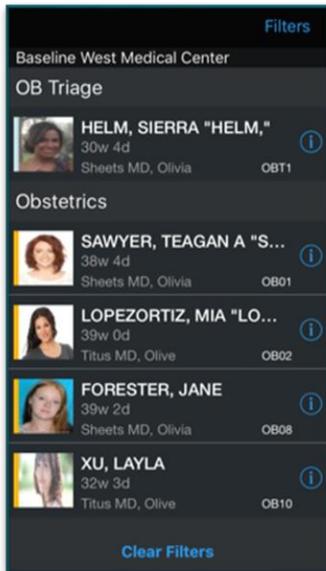

- Select the patient and view the patient's information based on the nursing documentation when the patient's name is selected.

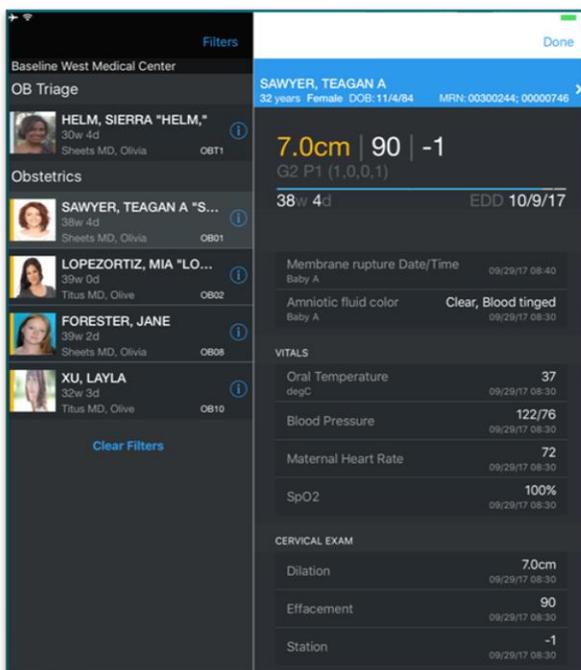


- From the lower portion of the screen, select **Monitor, Review and Order**.
 - Monitor

- From the monitor screen, the strip is viewable.



- Select ALL on annotations to view all the annotations for the patient.

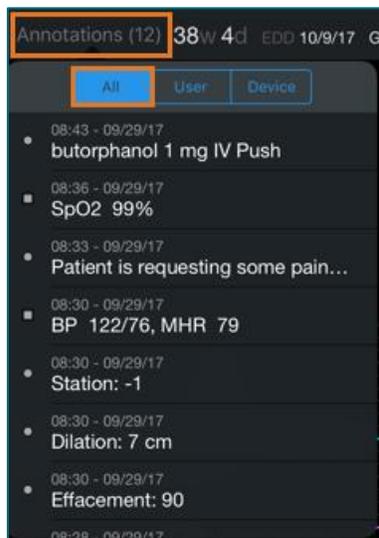


- To view a specific annotation, click the icon on the strip.




- Patient Information

- By selecting the banner bar for the patient, demographic information is available to view.
- Available information includes Demographics, encounter details, family contacts and preferred pharmacy

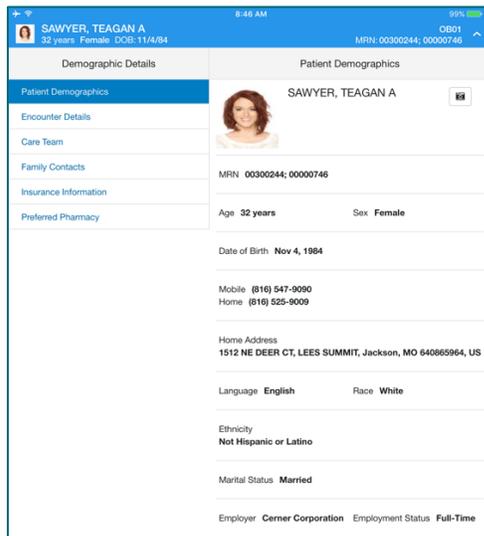

- Review

- Select review from the lower portion of the screen in order to review information from the pregnancy overview component and other pertinent patient information.

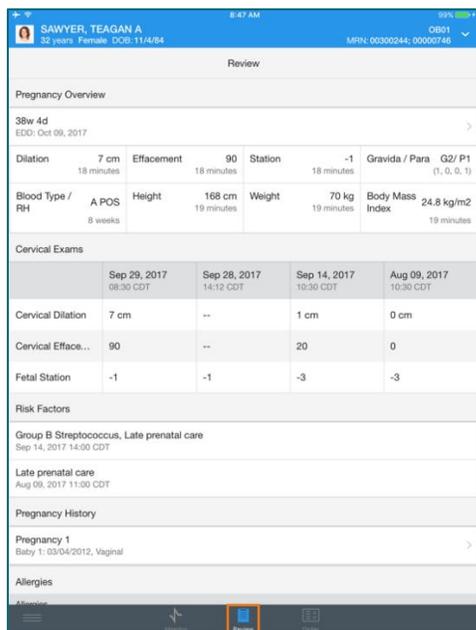

- To see more details, a lab result can be highlighted.

Automated Hematology	Latest	Previous
WBC (4.0-11.0) x10 ³ /mcL	6.3 x10 ³ /mcL 27 minutes	6.0 x10 ³ 8 weeks
RBC (3.80-4.80) x10 ³ /mcL	4.20 x10 ³ /mcL 27 minutes	4.00 x10 ³ 8 weeks
Hgb (12.0-16.0) g/dL	12.3 g/dL 27 minutes	12.0 % 8 weeks
Hct	38.0 %	37.0 %

Hgb
(12.0-16.0) g/dL
Trend

12.3 g/dL
Sep 29, 2017 8:22 a.m.

12.0 %
Aug 9, 2017 10:30 a.m.

- Order
 - Orders can be viewed by selecting **Order** at the bottom of the screen.
 - Orders will be displayed, to view more details, select the order name.



NOTE: The app is a view only application to view patient's fetal monitoring status.

For questions regarding process and/or policies, please contact your unit's Clinical Educator. For questions regarding workflow, please [place a ticket](#) to Health Informatics. For any other questions please contact the Customer Support Center at: 207-973-7728 or 1-888-827-7728.