July 9, 2025



FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in acute and ambulatory settings. This flyer will discuss the use of FetaLink +.

FetaLink +

FetaLink+ is a mobile application that will be used by OB providers on their IOS mobile devices (phone or iPad). The app is a view only mode for the patient's fetal monitoring.


From the app store, select FetaLink+ and install on an IOS device.




• When the app launches, sign in with Cerner login and password at the prompt.




• Filters can be created by location and providers.

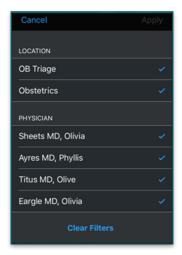

• The patient's names with location. EGA, room number, and provider's name will be displayed based on location.

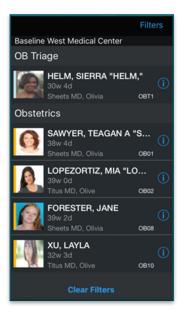

• Select the patient and view the patient's information based on the nursing documentation when the patient's name is selected.

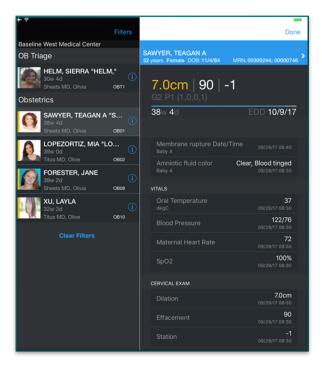

- From the lower portion of the screen, select **Monitor**, **Review** and **Order**.
 - Monitor

• From the monitor screen, the strip is viewable.



• Select **ALL** on annotations to view all the annotations for the patient.




• To view a specific annotation, click the icon on the strip.



Patient Information

- By selecting the banner bar for the patient, demographic information is available to view.
- Available information includes Demographics, encounter details, family contacts and preferred pharmacy

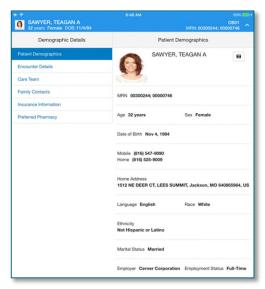

Review

• Select review from the lower portion of the screen in order to review information from the pregnancy overview component and other pertinent patient information.



• To see more details, a lab result can be highlighted.



Order

- Orders can be viewed by selecting **Order** at the bottom of the screen.
- Orders will be displayed, to view more details, select the order name.



NOTE: The app is a view only application to view patient's fetal monitoring status.