



FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in acute and ambulatory settings. This flyer will discuss the use of FetaLink+.

FetaLink+

FetaLink+ is a mobile application that will be utilized by OB providers on their IOS mobile devices (phone or iPad). This app is a view only mode for the patient's fetal monitoring.

➤ From the app store, select FetaLink+ and install on your IOS device.

- Open the FetaLink+ app.
- Log in with your username and password.
Touch Login.

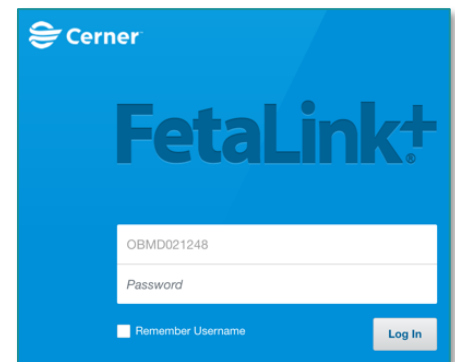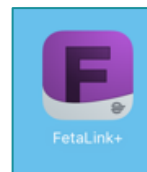

➤ Filters can be created by location and providers.

- The patient's names with location, EGA, room number, and provider's name will be displayed based on location.
- Select the patient and view the patient's information based on the nursing documentation when the patient's name is selected.





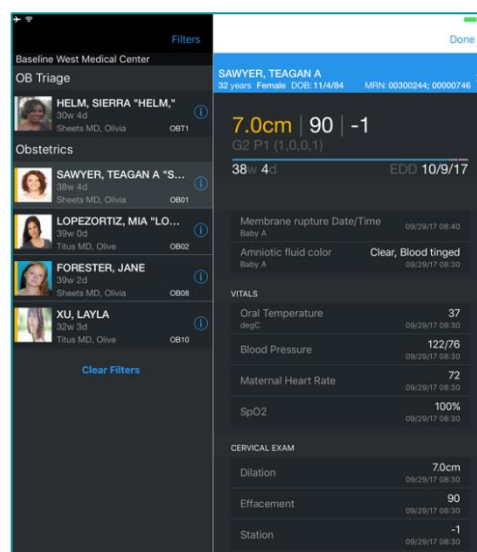


Chart Review Options

From the lower portion of the screen, you can select Monitor, Review and Order.


➤ Monitor

- From the monitor screen, the strip is viewable.
- By clicking on **Annotations**, you can view all the annotations on the patient.
 - To view a specific annotation, you can click on the icon on the strip at that time.

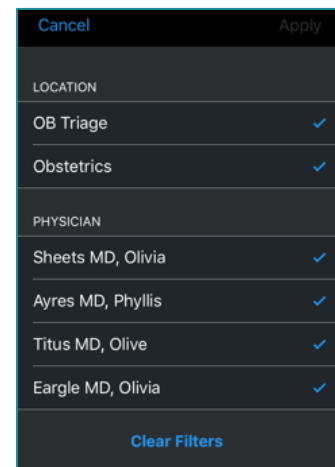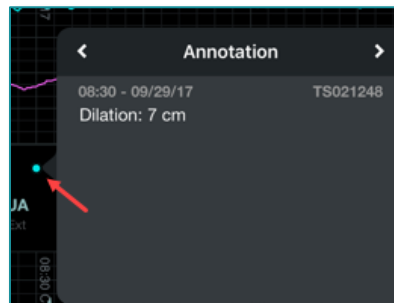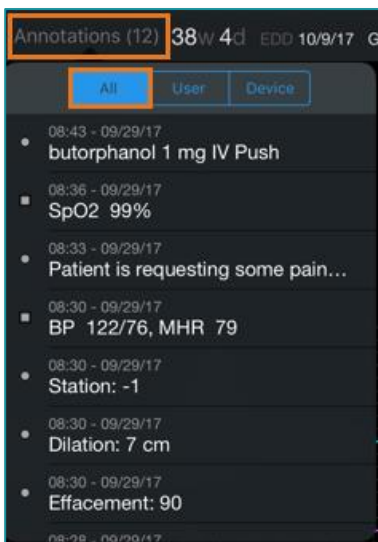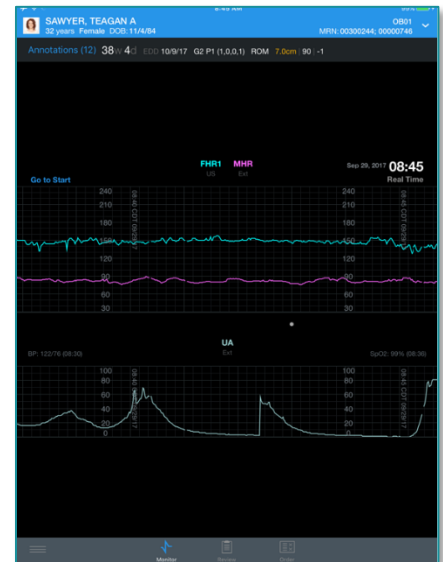

➤ Patient Information

- By selecting the banner bar for the patient, demographic information is available to view.
- Available information includes demographics, encounter details, family contacts and preferred pharmacy.

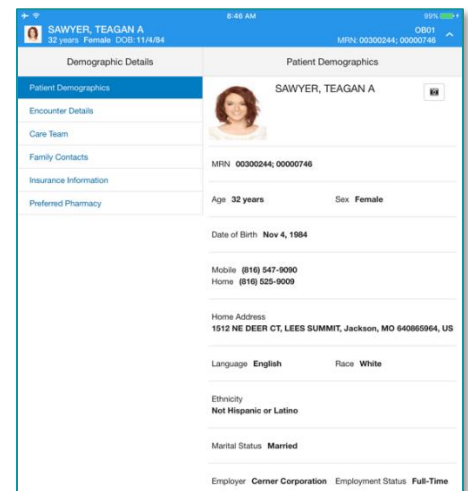

➤ Review


- By selecting **Review** from the lower portion of your screen, you can review information from the Pregnancy Overview component and other pertinent patient information.
- To view more details, a lab result can be highlighted.

Labs (Last 3 Years)		
Automated Hematology	Latest	Previous
WBC (4.0-11.0) x10 ³ /mcL	6.3 x10 ³ /mcL 27 minutes	6.0 x10 ³ 8 weeks
RBC (3.80-4.80) x10 ³ /mcL	4.20 x10 ³ /mcL 27 minutes	4.00 x10 ³ 8 weeks
Hgb (12.0-16.0) g/dL	12.3 g/dL 27 minutes	12.0 % 8 weeks
Hct	38.0 %	37.0 %

Hgb
(12.0-16.0) g/dL
Trend

12.3 g/dL
Sep 29, 2017 8:22 a.m.

12.0 %
Aug 9, 2017 10:30 a.m.

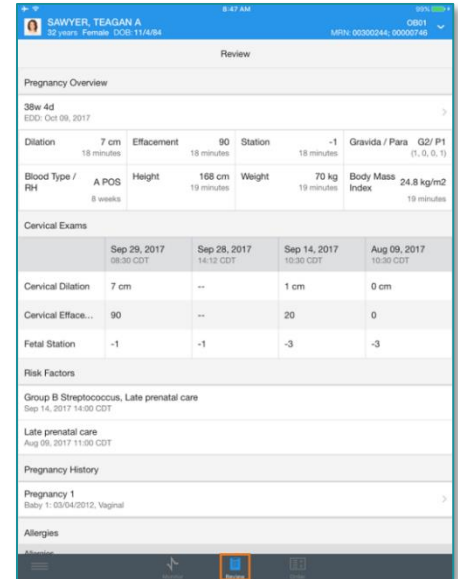

➤ Order

- Select **Order** at the bottom of the screen to view the patient's orders.
- Orders will be displayed. To view more details, select the Order name.

NOTE: This app is a view only application to view your patient's fetal monitoring status.

NOTE: If you are not currently enrolled in Cerner Mobile, you will need to obtain an access code from Security.

