

This Quick Reference Guide (QRG) demonstrates how to compare studies in Enterprise Imaging.

Compare Studies

➤ From the Report tab of a study:

NOTE: The Eyeball icon indicates there are images to review for the study.

- The icon indicates a study is an Active study.
- The X icon means there are no images available for a Comparison study.
- The Report icon indicates there is a report available for a Comparison study.
- The Comment box icon indicates there are study comments attached to a Comparison study.



STEP 1: Double-click the **Comparison study** to compare with the Active study.

- The right pane updates to display the Report information of the selected Comparison study.


STEP 2: Click the **Image area** icon.

- The studies display.



NOTE: Report icons with a green line below them indicate this is an image from the Active study.

- The white Report icon with a purple line below indicates this an image from a Prior study (the Comparison study).
- When viewing Comparison studies, a red border displays around the image in context. In the Clinical sidebar, the thumbnail for the image also displays with a red border, and the thumbnail image not in context displays with a dotted red border.

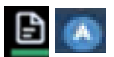

STEP 3: Click the **Hanging protocols** icon.

- The Hanging protocols window displays.



STEP 4: Click the **star** of an option to select it as the favorite view option.

- The star turns yellow to indicate it is now the favorite view.



NOTE: Making the view a favorite does not make it the default view.

STEP 5: Click-and-drag a view option to the far left position to set it as the default view.

STEP 6: Click a view option to update the screen with the selected Hanging protocol view.

STEP 7: Double-click an image to view it in full screen.

STEP 8: Double-click the image once more to return to the selected Hanging protocol view.

STEP 9: Click the **patient banner** to view a list of Comparison studies in the Image area view.

TESTING, ALEXIS MRN 2340823 - NLH

- A window displays the Active and Comparison study panes.

STEP 10: Double-click a different Comparison study to compare it to the Active study or click anywhere outside the window to close it.

STEP 11: Use the mouse wheel to scroll through the images with the red border.

STEP 12: Click an image in the right pane to bring the Comparison study images into context.

STEP 13: Use the **Enable/disable navigation autolink** icon to link two images from different studies together.

