

---

This Quick Reference Guide (QRG) demonstrates how to compare studies in Enterprise Imaging.

---

## Compare Studies

From the **Report** tab of a study:

**NOTE:** The Eyeball icon indicates there are images to review for the study. 

The icon indicates a study is an Active study. 


The X icon means there are no images available for a Comparison study.

The Report icon indicates there is a report available for a Comparison study. 

The Comment box icon indicates there are study comments attached to a Comparison study. 

**STEP 1:** Double-click the Comparison study to compare with the Active study.

- The right pane updates to display the Report information of the selected Comparison study.

**STEP 2:** Click the Image area icon. 

- The studies display.


**NOTE:** Report icons with a green line below them indicate this is an image from the Active study.  

The white Report icon with a purple line below indicates this an image from a Prior study (the Comparison study). 

When viewing Comparison studies, a red border displays around the image in context. In the Clinical sidebar, the thumbnail for the image also displays with a red border, and the thumbnail image not in context displays with a dotted red border.

**STEP 3:** Click the Hanging protocols icon. 

- The Hanging protocols window displays.

**STEP 4:** Click the star of an option to select it as the favorite view option. 

- The star turns yellow to indicate it is now the favorite view.

**NOTE:** Making the view a favorite does not make it the default view.

**STEP 5:** Click-and-drag a view option to the far left position to set it as the default view.

**STEP 6:** Click a view option to update the screen with the selected Hanging protocol view.

**STEP 7:** Double-click an image to view it in full screen.

---

**STEP 8:** Double-click the image once more to return to the selected Hanging protocol view.

**STEP 9:** Click the patient banner to view a list of Comparison studies in the Image area view.

TESTING, ALEXIS MRN 2340823 - NLH

- A window displays the Active and Comparison study panes.

**STEP 10:** Double-click a different Comparison study to compare it to the Active study or click anywhere outside the window to close it

**STEP 11:** Use the mouse wheel to scroll through the images with the red border.

**STEP 12:** Click an image in the right pane to bring the Comparison study images into context.

**STEP 13:** Use the **Enable/disable navigation autolink** icon to link two images from different studies together.

