


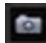


This Quick Reference Guide (QRG) demonstrates the basic use of Multiple Planar Reconstruction (MPR) features in Enterprise Imaging.

Use MPR Features

From the Search screen with results below:

- STEP 1:** Double-click the appropriate study.
- The Text area screen displays, open to the orders tab of the study.
- STEP 2:** Click the **Image area** icon. 
- The Image area displays the images for the study in the default Hanging protocol view.
- STEP 3:** Hover over the bottom center of the viewport of the appropriate image.
- An Up Arrow displays.
- STEP 4:** Hover over the **Up Arrow**. 
- A toolbar displays icons and options for the image.
- STEP 5:** Hover over the current view sub option.
- The View menu displays.
- STEP 6:** Click **MPR**.
- The Orientation menu displays.
- STEP 7:** Click the appropriate MPR orientation option.
- The screen updates to display the selected view for the image.
- STEP 8:** Use the **Pushpin** icon to pin the toolbar open in the viewport. 
- STEP 9:** Use the **Capture** icon to take a secondary capture of the image. 

NOTE: Viewport only, Capture all screens, or Save the series as a movie can all be captured.

STEP 10: Use the **Hanging protocols** icon to select the format to view the MPR images. 

NOTE: Select the view from the MPR MIP 3D window.