
This Quick Reference Guide (QRG) demonstrates the basic use of Multiple Planar Reconstruction (MPR) features in Enterprise Imaging.

Use MPR Features

➤ From the Search screen with results below:

STEP 1: Double-click the appropriate study.
▪ The text area screen displays, open to the orders tab of the study.

STEP 2: Click the **Image area** icon. 
▪ The image area displays the images for the study in the default Hanging protocol view.

STEP 3: Hover over the bottom center of the viewport of the appropriate image.
▪ An up arrow displays.

STEP 4: Hover over the **Up Arrow**. 
▪ A toolbar displays icons and options for the image.

STEP 5: Hover over the current view sub option.
▪ The View menu displays.

STEP 6: Click **MPR**.
▪ The Orientation menu displays.

STEP 7: Click the appropriate MPR orientation option.
▪ The screen updates to display the selected view for the image.

STEP 8: Use the **Pushpin** icon to pin the toolbar open in the viewport. 

STEP 9: Use the **Capture** icon to take a secondary capture of the image. 

NOTE: **Viewport only, Capture all screens, or Save the series as a movie can all be captured.**

STEP 10: Use the **Hanging protocols** icon to select the format to view the MPR images. 

NOTE: **Select the view from the MPR MIP 3D window.**