
This Quick Reference Guide (QRG) demonstrates how to manage measurements in Enterprise Imaging.

Manage Measurements

Add Measurements to Report

➤ From the Image area screen of a patient's study:

STEP 1: Right-click the measurement to view the options.

NOTE: Choose to add to current report section, add to measurement section, or add all measurements to measurement section.

STEP 2: Click anywhere in the Image area outside the menu to close it.

Delete Measurement

➤ From the Image area screen of a patient's study:

STEP 1: Click the measurement line.

NOTE: Click the measurement line, not the measurement itself.


STEP 2: Right-click the measurement line; then click **Delete**.

Secondary Capture


➤ From the Image area screen of a patient's study:

STEP 1: Hover over the bottom center of the viewport.

- The Up Arrow displays. 

STEP 2: Hover over the Up Arrow. 

- The toolbar menu displays.

STEP 3: Click the Capture icon. 

NOTE: The Viewport only option takes a secondary capture of the image in the viewport only and sends just that image to the report.

The All screens option takes secondary captures of all the images in the study and sends all the images to the report.