

---

This Quick Reference Guide (QRG) demonstrates how to manage measurements in Enterprise Imaging.

---

## Manage Measurements

Add Measurements to Report

➤ From the Image area screen of a patient's study:

**STEP 1:** Right-click the measurement to view the options.

**NOTE:** Choose to add to current report section, add to measurement section, or add all measurements to measurement section.

**STEP 2:** Click anywhere in the Image area outside the menu to close it.

## Delete Measurement

➤ From the Image area screen of a patient's study:

**STEP 1:** Click the measurement line.

**NOTE:** Click the measurement line, not the measurement itself.

**STEP 2:** Right-click the measurement line; then click **Delete**.

## Secondary Capture

➤ From the Image area screen of a patient's study:

**STEP 1:** Hover over the bottom center of the viewport.

- The **Up Arrow** displays.



**STEP 2:** Hover over the **Up Arrow**.

- The toolbar menu displays.



**STEP 3:** Click the **Capture** icon.



**NOTE:** The **Viewport only** option takes a secondary capture of the image in the viewport only and sends just that image to the report.

The **All screens** option takes secondary captures of all the images in the study and sends all the images to the report.