

---

The Horus Scope is a peripheral designed to work with the Amwell cart. It is intended to take digital photographs and video of the eye fundus, ears, and skin.

---

### Horus Scope Connection to Amwell Cart

- Connect the mini-USB end to the bottom of the Horus Scope handle or to the charging base.

**NOTE:** If there is a charging base, will need to plug the mini-USB to the bottom of the Horus scope when using the device.

- On the other end of the charging cable, take the regular size USB and insert it into any USB port of the Amwell cart.

**NOTE:** If there are two USB plugs, plug in the USB with two wires connected to it. The single-wire USB will not allow video to be shared.

- Check if the **UVC mode** is **enabled**.
  - Tap the **camera** button in the top left corner of the touchscreen.
  - Tap the **menu** button that appears in the bottom left corner.
  - Tap the **UVC** button in the top right corner of the screen, will be **highlighted blue** once enabled.

### Horus Scope Connection to Amwell Cart

- On the **Amwell cart** tap the **Menu** button.
- Tap **Devices**.
  - The Horus scope will show as an available device.
- Tap **Share**.
  - The video will show on the Amwell cart and the provider will be able to view the video.
  - Images cannot be saved at this time.
- Tap **End Share** to stop sharing the Horus scope's image.

*Content created by Amwell Clinical Support  
Content last reviewed January 15, 2021*

---

For questions regarding process and/or policies, please contact your unit's Clinical Educator. For questions regarding workflow, please [place a ticket](#) to Health Informatics. For any other questions please contact the Customer Support Center at: 207-973-7728 or 1-888-827-7728.

---