


# 5 simple steps for creating a POCUS report


Be sure to consult the full instructions for use prior to using the device.

1. Select a **POCUS Exam(s)** from the dropdown list.
2. Complete the report template using either **Quick Reports** or manually fill in fields.
3. Enter **Comments** and **Follow-up** as needed.
4. Verify **Operators** are entered and correct.
5. Upon completion of the report, change the exam status to close report.

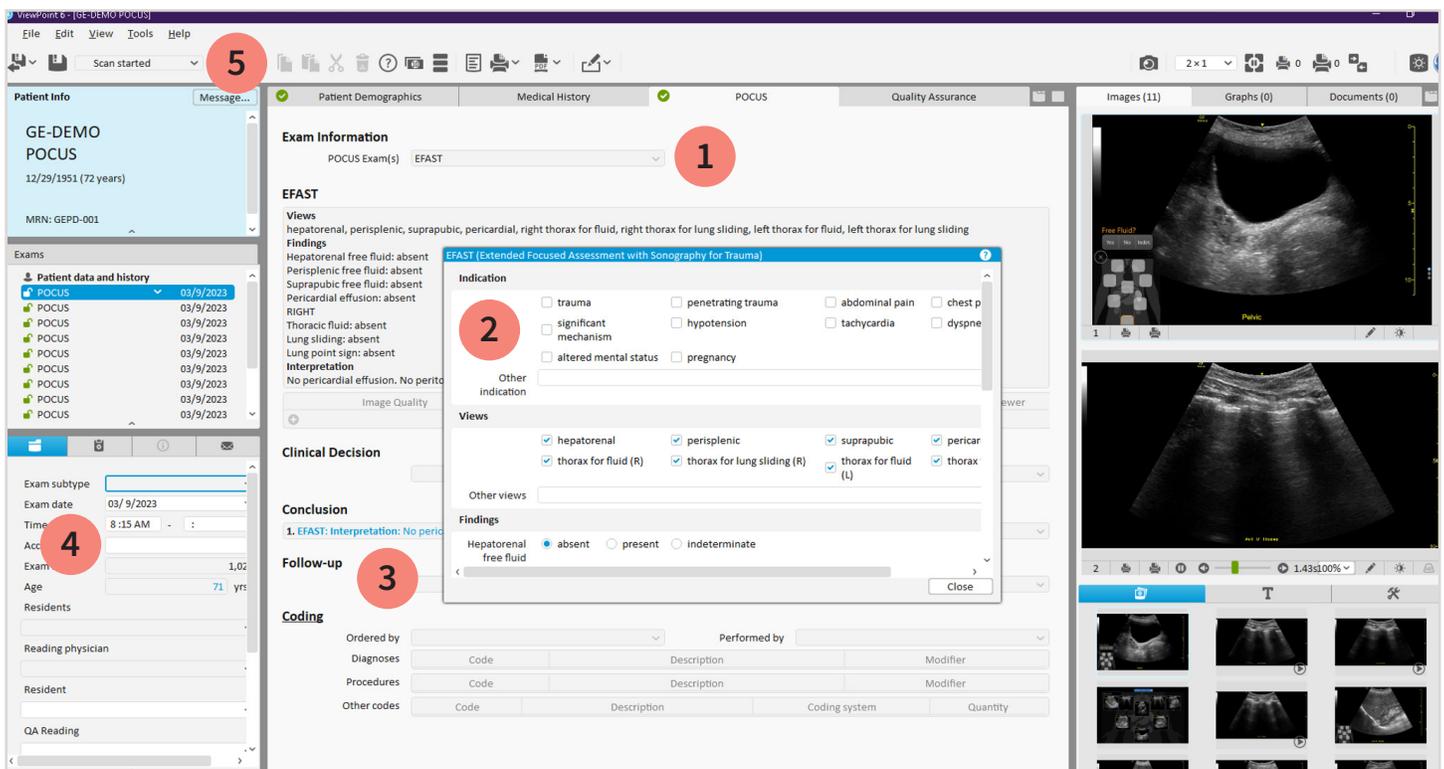


## 5 simple steps for creating a ViewPoint 6 Report on the Venue™ system

1. After the scan, click the **Report** icon and select **VP Report**.
2. From the list, click the exam type and then **Select**.
3. Fill in the report. You can either scroll to each section or use the tabs on the left of the screen.
4. When the report is complete, hit **End Exam** to close.
5. Once you closed the exam on the Venue system, you can still review, edit and finish your report in ViewPoint 6.

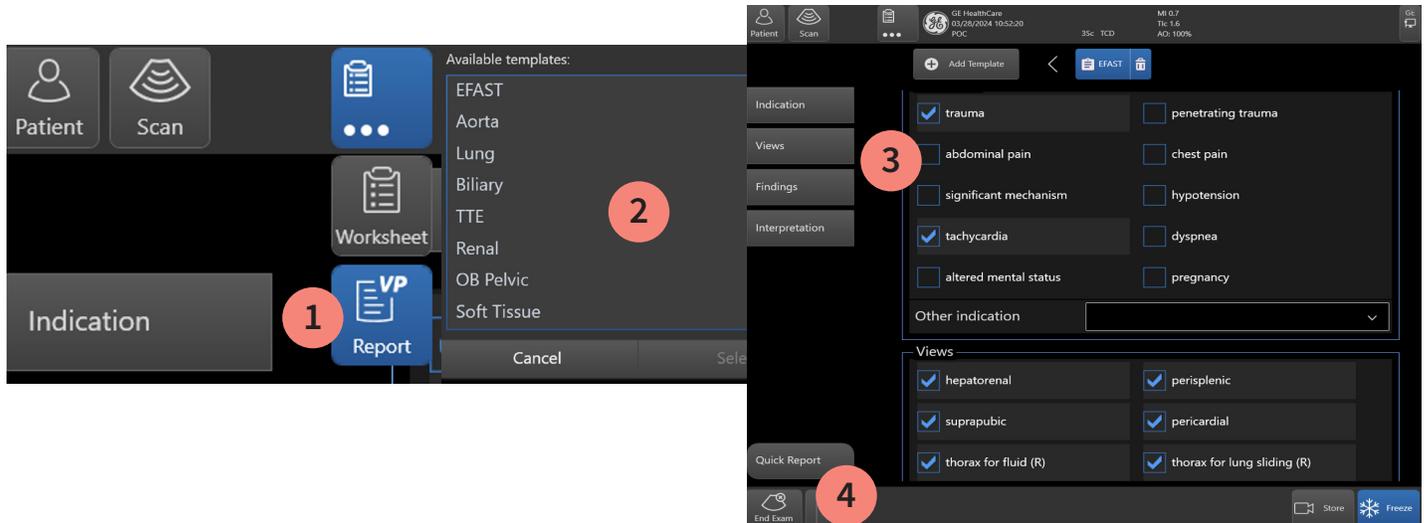

## 5 simple steps for creating a ViewPoint 6 Report on MobileConnect

1. After the scan, select the patient name and click **Load**.
2. When the patient is loaded, click **+ Add exam** and select the appropriate exam and **OK**.
3. Complete the report. You can add additional templates if needed, or click the arrow to advance.
4. When the report is complete, hit **Complete** to close.
5. Once you closed the exam on the Venue system, you can still review, edit and finish your report in ViewPoint 6.

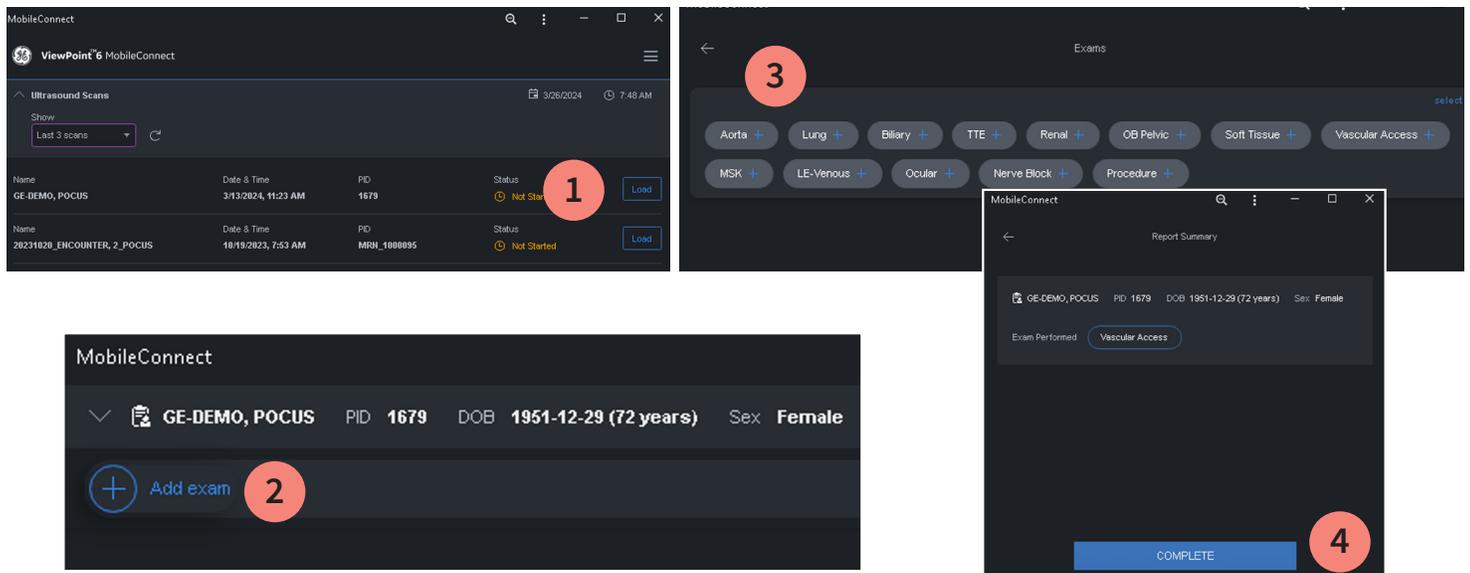

Be sure to consult the full instructions for use prior to using the device.

Products mentioned in the material may be subject to government regulations and may not be available in all countries. Shipment and effective sale can only occur after approval from the regulator. Please check with local GE HealthCare representative for details.

© 2024 GE HealthCare. ViewPoint and Venue are trademarks of GE HealthCare. GE is a trademark of General Electric Company used under trademark license.

April 2024 | JB28858XX